

Calcium P FS

Prestige 24i

Order information

Cat. No. 1 1181

Notes

1. Please refer to the package insert for Calcium P FS for detailed information about the test on the following: Clinical Relevance, Method and Principle, Composition and Stability of the Reagents, Specimens, Calibrators and Controls, Performance Characteristics regarding Measuring Range / Specificity/Interferences / Sensitivity/Limit of Detection / Precision (Reproducibility, Repeatability) / Method Comparison / Reference Ranges / Literature
2. The stability of the reagent on board the analyser is 8 weeks provided that contamination and evaporation are avoided.
3. Manufactured by: DiaSys Diagnostic Systems GmbH
Alte Strasse 9, 65558 Holzheim, Germany.

****This application was set up and evaluated at DiaSys. It is based on the standard equipment at that time and does not apply to any equipment modifications undertaken by the manufacturer or by other persons.**

Item Name	#	CaP
-----------	---	-----

<p>DATA INFORMATION</p> <table border="1" style="width: 100%; border-collapse: collapse;"> <tr> <td style="width: 60%;">UNITS</td> <td>mmol/l</td> </tr> <tr> <td>DECIMALS</td> <td>2</td> </tr> </table>	UNITS	mmol/l	DECIMALS	2	<p>CALIBRATION</p> <table border="1" style="width: 100%; border-collapse: collapse;"> <tr> <td style="width: 50%;">TYPE</td> <td colspan="3">Linear</td> </tr> <tr> <td colspan="4">STANDARD</td> </tr> <tr> <td>#1</td> <td>*</td> <td>#4</td> <td></td> </tr> <tr> <td>#2</td> <td></td> <td>#5</td> <td></td> </tr> <tr> <td>#3</td> <td></td> <td>#6</td> <td></td> </tr> </table>	TYPE	Linear			STANDARD				#1	*	#4		#2		#5		#3		#6																								
UNITS	mmol/l																																															
DECIMALS	2																																															
TYPE	Linear																																															
STANDARD																																																
#1	*	#4																																														
#2		#5																																														
#3		#6																																														
<p>ANALYSIS</p> <table border="1" style="width: 100%; border-collapse: collapse;"> <tr> <td style="width: 60%;">TYPE</td> <td>END</td> </tr> <tr> <td>Main W.Length 1</td> <td>660</td> </tr> <tr> <td>Main W.Length 2</td> <td>800</td> </tr> <tr> <td>METHOD</td> <td>DIASYS</td> </tr> </table>	TYPE	END	Main W.Length 1	660	Main W.Length 2	800	METHOD	DIASYS	<p>NORMAL RANGE</p> <table border="1" style="width: 100%; border-collapse: collapse;"> <thead> <tr> <th rowspan="2"></th> <th colspan="2">MALE</th> <th colspan="2">FEMALE</th> </tr> <tr> <th>LOW</th> <th>HIGH</th> <th>LOW</th> <th>HIGH</th> </tr> </thead> <tbody> <tr> <td>Serum</td> <td>#</td> <td>#</td> <td>#</td> <td>#</td> </tr> <tr> <td>Urine</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Plasma</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>CSF</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Dialysis</td> <td></td> <td></td> <td></td> <td></td> </tr> <tr> <td>Other</td> <td></td> <td></td> <td></td> <td></td> </tr> </tbody> </table>		MALE		FEMALE		LOW	HIGH	LOW	HIGH	Serum	#	#	#	#	Urine					Plasma					CSF					Dialysis					Other				
TYPE	END																																															
Main W.Length 1	660																																															
Main W.Length 2	800																																															
METHOD	DIASYS																																															
	MALE		FEMALE																																													
	LOW	HIGH	LOW	HIGH																																												
Serum	#	#	#	#																																												
Urine																																																
Plasma																																																
CSF																																																
Dialysis																																																
Other																																																
<p>CORR.</p> <table border="1" style="width: 100%; border-collapse: collapse;"> <tr> <td style="width: 10%;">Y =</td> <td style="width: 20%; text-align: center;">SLOPE</td> <td style="width: 10%;"></td> <td style="width: 20%; text-align: center;">INTER</td> <td style="width: 10%;"></td> <td style="width: 10%;"></td> </tr> <tr> <td></td> <td style="text-align: center;">1</td> <td style="text-align: center;">X +</td> <td style="text-align: center;">0</td> <td></td> <td></td> </tr> </table>		Y =	SLOPE		INTER				1	X +	0																																					
Y =	SLOPE		INTER																																													
	1	X +	0																																													

Page : 1

*input calibrator value
entered by the user

Item Name # CaP

ASPIRATION

KIND Single Double

SAMPLE	3	μl
REAGENT1 VOL	200	
REAGENT2	50	

Third Mix. OFF ON

R1 Blank Water Blk R1 Blank1

MONITOR

0 LEVEL POINT	1
SPAN	3

DATA PROCESS

READ

	START	END
MAIN	36	36
SUB	30	31

ABSORBANCE LIMIT

LOW	0.000
HIGH	3.000

FACTOR

Blk correction	0.80237
----------------	---------

ENDPOINT LIMIT 2.000

LINEAR CHECK(%) 0

Dilution

Diluent 99:Dil1 100:Dil2

PROZONE CHECK

	START	END	LIMIT(%)	
FIRST				
SECOND				<input type="radio"/> Low <input type="radio"/> High
THIRD				<input type="radio"/> Low <input type="radio"/> High

Page : 2

Item Name # CaP

Auto Rerun SW

On Off

Auto Rerun Range (Result)

	<input type="radio"/> On <input checked="" type="radio"/> Off	<input checked="" type="radio"/> On <input type="radio"/> Off
	Lower	Higher
Serum		6.24
Urine		
Plasma		
CSF		
Dialysis		
Other		

Auto Rerun Condition (Absorbance)

Absorbance Range

Lower On Off

Higher On Off

Prozone Range On Off

Page : 3

*input calibrator value
entered by the user